

How to administer



1 Preparation

Administer a **topical anaesthetic agent** into the conjunctival sac, following the product manufacturer's instructions.



Ensure the canaliculus is open and suitable for Lacrifill® administration by performing **punctal irrigation**, confirming good lacrimal drainage.

2 Set-Up

Remove the syringe from the packaging and **remove the cap** from the end of the syringe.



Fit the accompanying cannula onto the end of the syringe, ensuring it is secured as **tightly as possible**.



With the syringe **facing upwards**, it is vital to **prime** the cannula by gently applying steady pressure until approximately **0.1 mL of gel** is expressed through the tip.



3 Administration

Lacrifill® can be administered at the **slit lamp** or inserted using **loupes** as the patient reclines in a chair.



Gently **pull down the lower eyelid** to expose the punctum and hold the eyelid firmly.

Carefully insert the cannula into the **inferior punctum**.

Apply **steady, gentle pressure** to the syringe, filling the canaliculus corresponding approximately **0.2 mL of gel**.

After completing the administration in the first eye:

Re-secure the cannula, then **re-prime** it by expressing an additional 0.1 mL of gel through the tip.

Repeat the administration process for the second eye.

The administration of Lacrifill® relies on 3 key steps to ensure a successful procedure and a comfortable experience for patients:



1 Secure

Ensure the cannula is firmly tightened onto the syringe.




2 Prime

Gently express 0.1 mL upward to prime before administration.




3 Deliver

Use smooth, even pressure to deliver 0.2 mL of gel into lower canaliculus

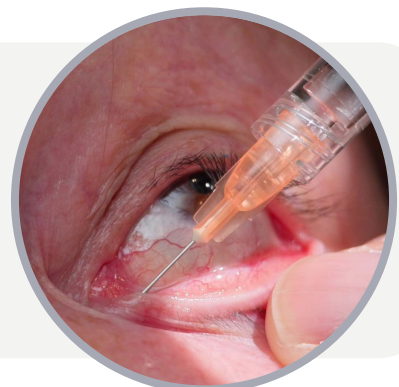

Scan this QR code to watch the Lacrifill® administration video

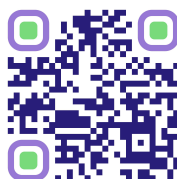

Scan this QR code for full Lacrifill® instructions for use

References

1. Lacrifill® Instructions for use.

GL-LAC-2600042 | June 2026